


EVOLUTION[®]

PATIENT POSITIONERS

Instructions for Use
Supine Positioners

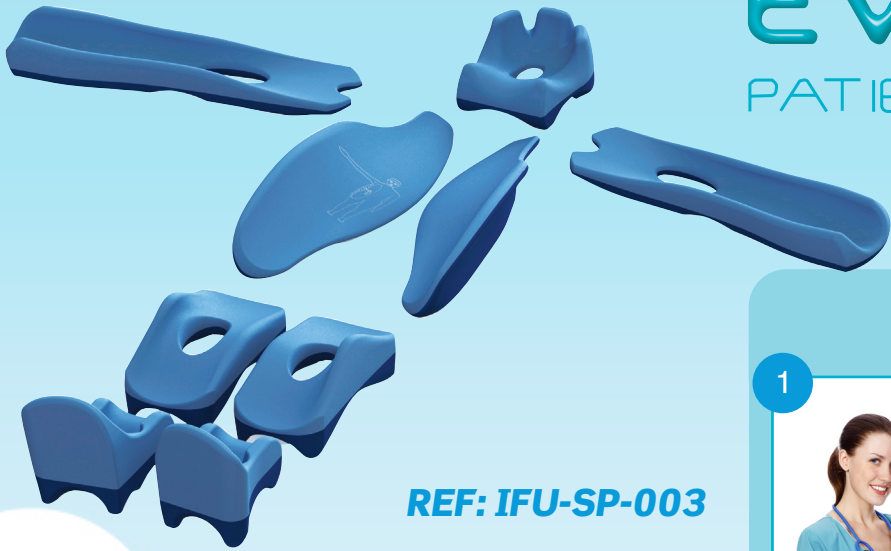

REF: IFU-SP-003

PRE - OP



HEAD - PAGE 3

SMALL MEDIUM LARGE

COVERS SUPINE HEAD POSITIONERS

REF CSHP12	REF SHP12 Small
REF CSHP14	REF SHP14 Large
REF CSHP23	REF SHP23 Medium
REF CSHP24	REF SHP24 Large



ARM - PAGE 4

MEDIUM LARGE

SUPINE ARM POSITIONER

REF	SAP14	LARGE
REF	SAP13	MEDIUM
STRAP AND RETAINER KIT		
REF	SRK14	LARGE



TORSO - PAGE 5

LARGE

SUPINE TORSO POSITIONERS

REF	STPL14	LEFT	LARGE
REF	STPR14	RIGHT	LARGE

STP14 LARGE PAIR RIGHT



KNEE AND FOOT - PAGE 6

MEDIUM LARGE


SUPINE FOOT POSITIONERS

REF	SFPL14	LARGE	Left
REF	SFPR14	LARGE	Right
REF	SFP13	MEDIUM	Universal
SUPINE KNEE POSITIONERS			
REF	SKP14	LARGE	Universal
REF	SKP13	MEDIUM	Universal



PRE-OP GENERAL

3 INSPECT BEFORE USE

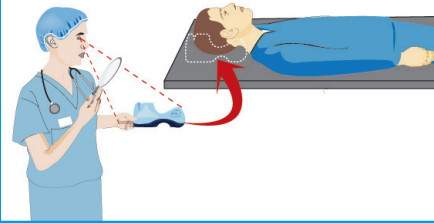

4 USE ON DRY SURFACE



5




6




7




8

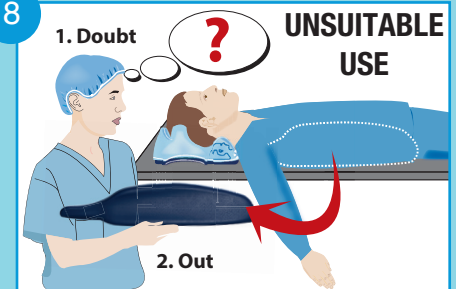

9

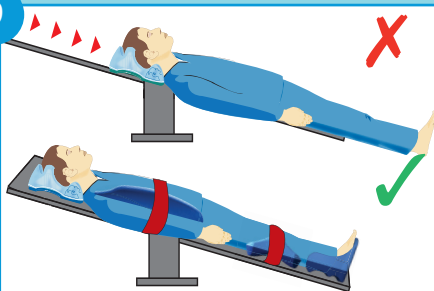


10

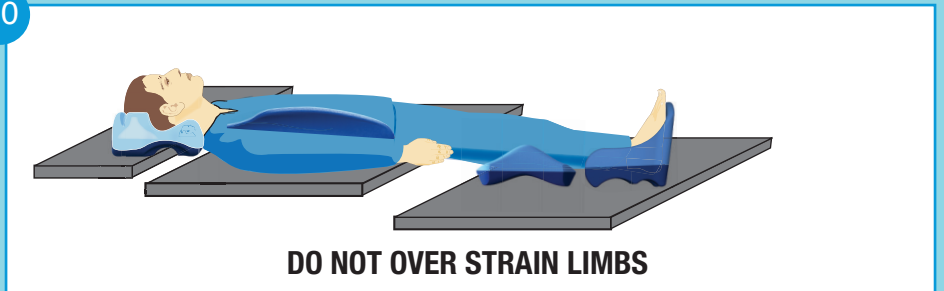

11

Avoid exaggerated stresses of positioners

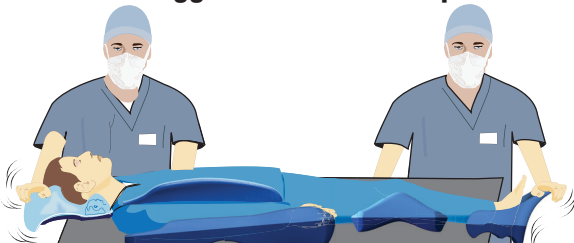



SMALL MEDIUM LARGE

Instructions for Use Supine Positioners **HEAD**

12 REF: IFU-SP-002

13

TSHP-01 x1
DTSHP-01 x50

14

15

16 DO NOT USE DAMAGED COVERS

17 PRODUCT INFORMATION

18

19

20

21

22

Keep Dry

23

24 DO NOT COVER APERTURE

25

26

27

28

29 PRODUCT MISUSE



MEDIUM/LARGE

Instructions for Use Supine Positioners **ARM**

30

REF: IFU-SP-002



31

SUPINE ARM POSITIONER



**MEDIUM SAPK13
LARGE SAPK14**

32

PATIENT HEIGHT & WEIGHT LIMITS

Medium
≥ 4'6"
≥ 137cm
Large
≥ 5'8"
≥ 173cm



BMI < 35
BMI=Body Mass Index

33

PRODUCT INFORMATION



34




35



ELBOW

36

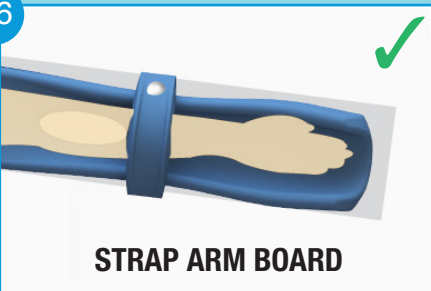


STRAP ARM BOARD

37




38

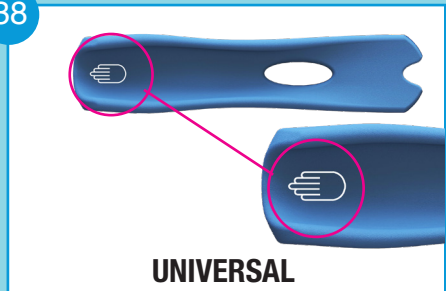

UNIVERSAL


39

PRODUCT MISUSE

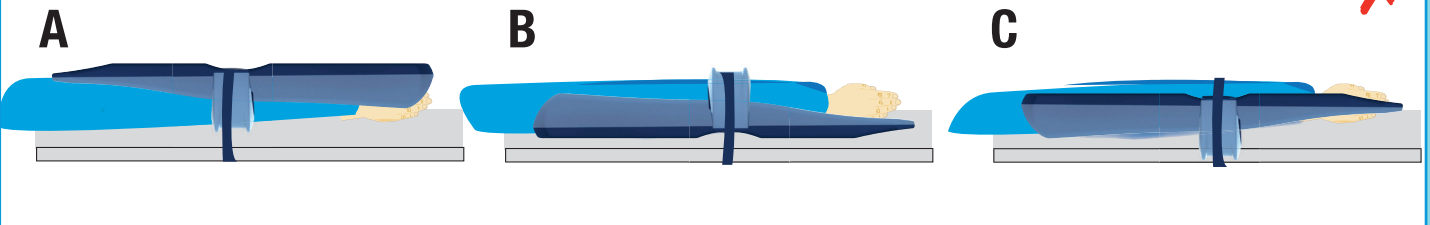

40

FASTEN

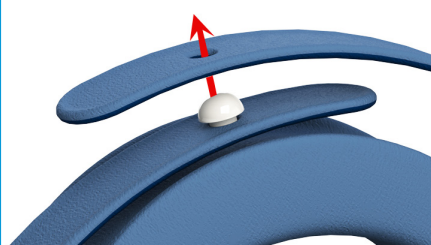



LARGE

Instructions for Use Supine Positioners **TORSO**

41

REF: IFU-SP-002




42

SUPINE TORSO POSITIONER




43

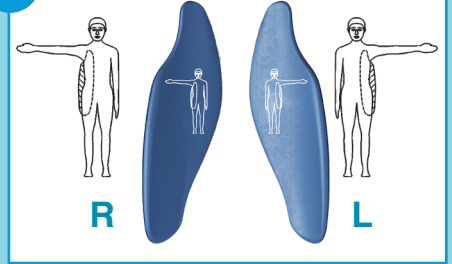

44

PATIENT HEIGHT & WEIGHT LIMITS

Large
≥ 5'8"
≥ 173cm

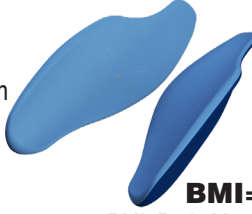


BMI < 35
BMI=Body Mass Index

45

PRODUCT INFORMATION



46

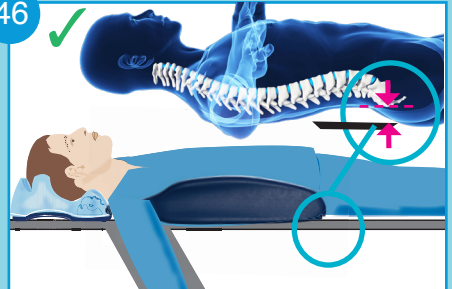

47





48



49

PRODUCT MISUSE

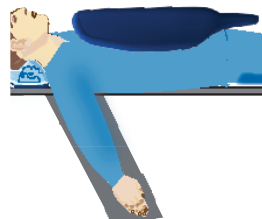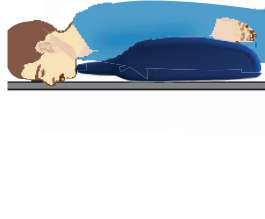

X



KNEE AND FOOT

50 REF: IFU-SP-002

51 SUPINE KNEE & FOOT POSITIONERS

LEFT SFPL14 - Large UNIVERSAL SKPP14 Large Pair

RIGHT SFPR14 - Large UNIVERSAL FPP13 Medium Pair SKPP13 Medium Pair

52

MEDIUM LARGE
 $\geq 20\text{cm}$ $\geq 20\text{cm}$
 $\geq 7.8''$ $\geq 7.8''$

53 PRODUCT INFORMATION

54

USE TOGETHER

55

\geq

56

57

58 PRODUCT MISUSE

A B C D

59

LARGE MEDIUM

Left Right Universal

60

61

UNIVERSAL




OPERATION


62




63

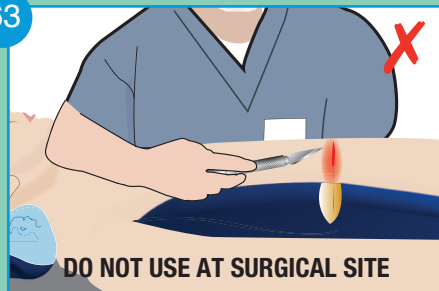


64

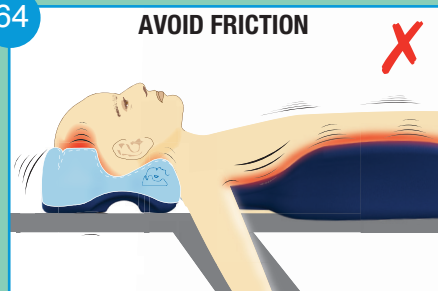


65

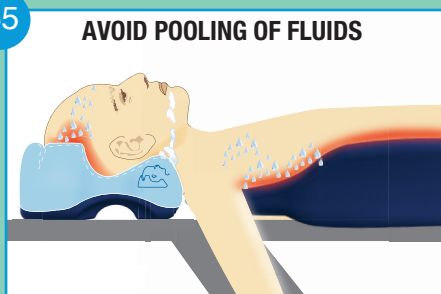

66

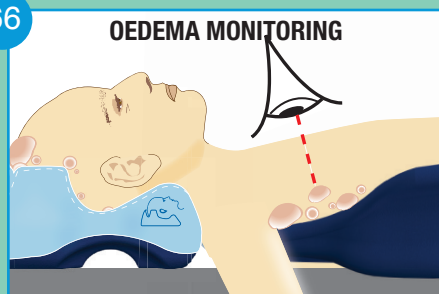

67

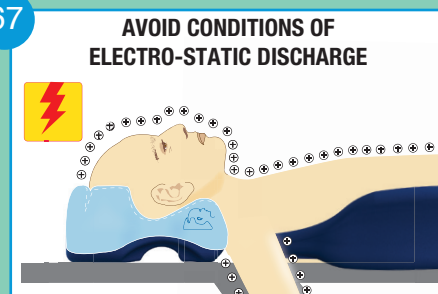

68




Enterprise & Research Incubation Centre,
Institute of Technology Carlow,
Kilkenny Road, Carlow,
Ireland.

t: 00353 (0)59 9175260

e: customercare@vascocare.com

w: www.vascocare.com


POST - OP

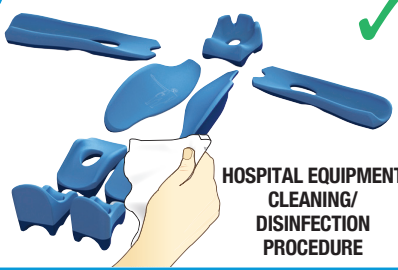
69  **HOSPITAL EQUIPMENT CLEANING/ DISINFECTION PROCEDURE**

70 **ACTIVE INGREDIENTS**

- Sodium Hypochloride
- Sodium Chloride
- Propan-1-ol (Alcohol Wipes)
- Sodium Dichloroisocyanurate

RECOMMENDED DISINFECTANT TO USE



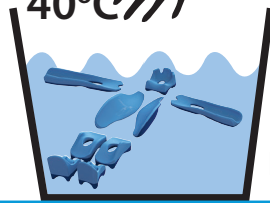





71 **104°F**
40°C 




Standard Detergent


72  **DISINFECTANT SOAK**

73


 MACHINE		 158°F 70°C SSS	 Sterilization GAS
			

74 **INSPECT AFTER USE**

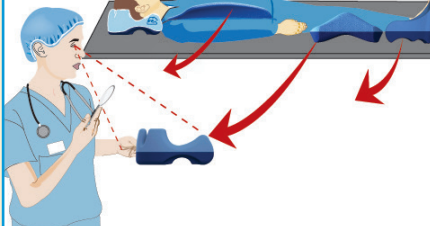

75  **DISINFECTANT STAINING OK**

76

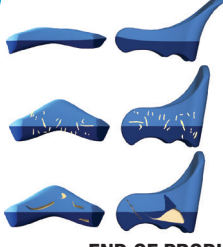


Deformed
Cracked
Torn


END OF PRODUCT LIFE


77 

Clinical waste

78 **+** **-** **Non-Conductive**

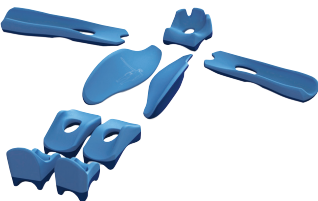

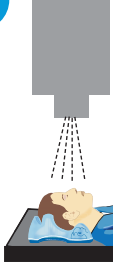
79  **PLAIN FILM X-RAY**

 **CT SCAN**

80 **NON STERILE**

81 **LATEX FREE**

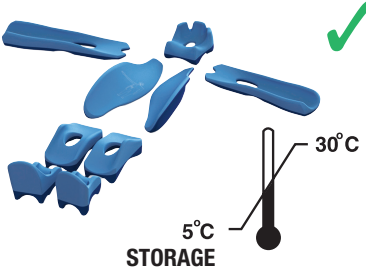

82 


5°C **30°C**

STORAGE

83 

Kg's

STORAGE

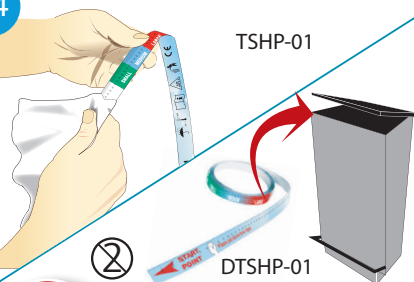84 

TSHP-01

DTSHP-01

85 

KEEP STOCK

86 



EVOLUTION Supine Positioners

Instructions for Use

Unique Device Identifier-Product Identifier (UDI-DI)

Supine Head Positioner Disposable Covers



CSHP12 Supine Head Positioner Disposable Cover, Small Style 1



CSHP23 Supine Head Positioner Disposable Cover, Medium Style 2



CSHP14 Supine Head Positioner Disposable Cover, Large Style 1



CSHP24 Supine Head Positioner Disposable Cover, Large Style 2

Supine Head Positioners



SHP12 Supine Head Positioner Small Style 1



SHP23 Supine Head Positioner Medium Style 2



SHP14 Supine Head Positioner Large Style 1



SHP24 Supine Head Positioner Large Style 2

Supine Arm Positioners



SAP13 Supine Arm Positioner Medium



SAPK13 Supine Arm Positioner Kit Medium



SAP14 Supine Arm Positioner Large



SAPK14 Supine Arm Positioner Kit Large



SRK14 Strap & Retainer Kit

EVOLUTION Supine Positioners

Instructions for Use

Unique Device Identifier-Product Identifier (UDI-DI)

Supine Torso Positioners



STPL14 Supine Torso Positioner
Left Large



STPR14 Supine Torso Positioner
Right Large



STP14 Supine Torso Positioner
Large Pair

Supine Foot Positioners



SFPP13 Supine Foot Positioner
Medium Pair



SFPL14 Supine Foot Positioner
Left Large



SFPR14 Supine Foot Positioner
Right Large



STP14 Supine Torso Positioner
Large Pair

Supine Knee Positioners



SKPP13 Supine Knee Positioner
Medium Pair



SKP14 Supine Knee Positioner
Large Single



SKPP14 Supine Knee Positioner
Large Pair

Any serious incident that has occurred in relation to this device should be reported to the Manufacturer and the Competent Authority of the Member State in which the user is established.